

## COPPER CONTROL LEVEL I-II

# En

**REF: VT-028 Copper Control Level I 5 x 4 mL**  
**REF: VT-029 Copper Control Level II 5 x 4 mL**

*Changes made in the instructions for use are marked as grey.*

Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package.

### INTENDED USE

Validity Copper Control Level I-II are for quality control of the Copper Liquid assay.

### CONTENTS / MATERIALS PROVIDED

Validity Copper Control Level I

**REF: VT-028**

Package: 5 x 4 mL **Liquid**

Validity Copper Control Level II

**REF: VT-029**

Package: 5 x 4 mL **Liquid**

### For use with;

Validity Copper Liquid Reagent

Validity Copper Calibrator

**VT-028 / VT-029** do not contain the human serum.

### Materials Required But Not Provided:

1. Class A volumetric pipette for liquid transfer
2. Distilled or deionized water meeting the specifications equivalent to USP (United States Pharmacopeial Convention) purified water.

### CONTROL STABILITY

Temperature-Conditions	Stability
Unopened at +2/+8°C	Expiry date on the vial.
Opened and stored at +2/+8°C	45 days

### PREPARATION OF CONTROL

Control is ready for use.

### INDICATIONS OF INSTABILITY OR DETERIORATION

Presence of extreme turbidity or microbial growth may indicate deterioration.

### PRECAUTIONS



Human source material. Treat as potentially infectious material. Each plasma donor used in the preparation of this product has been tested by an FDA-approved method and found negative for the presence of HIV 1/2 HBsAg, HCV, HIV-Ag antibodies. However, none of the known testing methods can offer complete assurance that the hepatitis B virus, Human Immunodeficiency Virus (HIV) or infectious agents are not present. All human-based products should be handled in accordance with Good Laboratory Practice (GLP) principles using appropriate precautions. Safety data sheets are available at [www.validity.com.tr](http://www.validity.com.tr) or you may contact your local representative.

### WARNINGS

IVD: For in Vitro Diagnostic use only.

Do not use expired reagents.

Reagents with two different lot numbers should not be interchanged.

For professional use.

Follow Good Laboratory Practice (GLP) guidelines.

Contains sodium azide.

**CAUTION:** Human source samples are processed with this product. All human source samples must be treated as potentially infectious materials and must be handled in accordance with OSHA standards.

### Danger

H317 :May cause allergic skin reaction.

### Precaution

P280 :Use protective gloves / clothes / glasses / mask.

P264 :Wash your hands properly after using.

P272 :Contaminated work clothes should not be allowed to be used outside of the workplace.

### Intervention

P302+P352 :Wash with plenty of water and soap if it contacts with skin.

P333+P313 :Seek medical help if it irritates your skin or develops rash.

## COPPER CONTROL LEVEL I-II

# En

P362+P364 :Remove contaminated clothes and wash properly before using.

### Disposal

P501 :Dispose the vials and contents according to the local regulations.

### REFERENCES

1. Burtis CA, Ashwood ER, Bruns DE, editors. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, 4<sup>th</sup> ed. St. Louis , MO, Elsevier Saunders; 2006:2263.
2. Occupational Safety and Health Standards: bloodborne pathogens. (29 CFR Part 1910.1030). Fed. Register. July 1, 2001; 17:260–273.
3. Directive 2000/54/EC. Official Journal of the European Communities No. L262 from October 17, 2000.
4. Tietz, N. W. (1983) Clinical guide to Laboratory Tests, p384 W.B. Saunders Co., Philadelphia.



**Archem Sağlık Sanayi ve Tic. A.Ş. (With official contract based manufacturing agreement with Validity Sağlık Hiz. Sanayi A.Ş. Company)**

Mahmutbey Mah. Halkalı Cad. No:124 Kat:4

Bağcılar/Istanbul/Turkey

Tel: + 90 212 444 08 92

Fax: +90 212 629 98 89

info@archem.com.tr www.archem.com.tr

info@validity.com.tr www.validity.com.tr



### SYMBOLS

**IVD**

In Vitro Diagnostic Medical Device

**LOT**

Lot Number

**CONTROL**

Control

**GTIN**

Global Trade Item Number

**REF**

Reference Number

**FOR USE WITH**

Identifies Products to Be Used Together

**PRODUCT OF TURKEY**

Product of Turkey



Manufacturer



Expiration Date



Temperature Limits



Consult Instructions for Use



Caution



Number of Tests